

BIO TEST

Your reliable home test kits series

Pregnancy Test Kit (midstream)




Product Code 9 555223 208012
For In-Vitro Diagnostic and self testing Use
Store at 2-30°C

INTENDED USE

BIOTEST Pregnancy Test Midstream (Urine) is a rapid chromatographic immunoassay for the qualitative detection of human chorionic gonadotropin (hCG) in urine to aid in the early detection of pregnancy.

HOW TO USE THE TEST

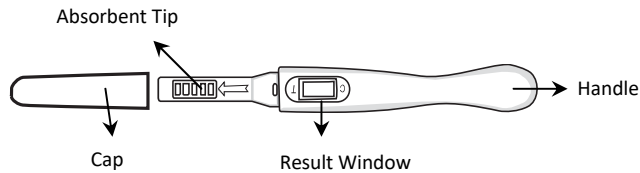

Step 1:

- Read instructions carefully before use.
- It is preferred to perform the test on the first morning urine because the first morning urine contains the highest amount of hCG.
- Check that the pouch is completely sealed then open the pouch.

Step 2:

- Remove the cap from the device.

Step 3:

- Hold the device by the handle with the exposed absorbing tip pointing downwards.
- Place the wick into the urine stream for at least 10 seconds; making sure that the entire tip is wet.
(Note: Do not urinate on the result window of the device. If you prefer, you can urinate into a clean and dry container, then dip only the absorbent tip of the midstream test into the urine for at least 10 seconds.)
- Remove the device from the urine stream (if you like, you may replace the cap - replacing the cap will not effect the results).

Step 4:

- Place the device on a flat surface with the window facing up. You will notice a pink color moving across the result window indicating that the test reaction is taking place.

Step 5:

- Read results within 5 minutes. Look for colored bands (lines) to appear in result window.
- Test results should not be read after 5 minutes.

HOW TO READ THE RESULTS

PREGNANT

Positive results: 2 bands.



The presence of two red lines in the result window means you are probably pregnant. One line may be lighter than the other; they do not have to match. Check with your doctor to confirm pregnancy.

NOT PREGNANT

Negative results : one band in the (C) area.



The presence of one red line in the control area (C) means you are probably not pregnant.

INVALID RESULT

Invalid results : no bands or 1 band in the (T) area.



The result is invalid if no red line appears in the control area (C), even if a line appears in the test area (T). You should repeat the test with a new test.

PRECAUTIONS & WARNINGS

1. Do not use the test after the expiration date.
2. If the package is not completely sealed do not use the test
3. The test should be performed in a well-lit area.
4. Use the test device immediately after opening it.
5. Do not touch the test window. This could affect results and may also impose personal hazards.
6. The pouch contains a desiccant pack to absorb humidity. Do not open the pack. Throw it away with the remaining of the test.
7. At the end of the test, wrap every thing you have used in a plastic bag and throw away in the bin. Do not forget to wash your hands properly.

CONTENTS

- Test Device
- Desiccant
- Instruction for Use

WHAT DOES THE RESULTS MEAN?

A positive result indicates that your urine contains hCG (the pregnancy hormone) at a concentration more than or equal to 25mIU/ml which is our device sensitivity and you can assume you are pregnant. You should consult your doctor who is best able to further advise you on proper prenatal care. If you are taking medication, inform your doctor immediately and make sure it is safe for your baby.

A negative result means that no hCG has been detected and you can assume you are not pregnant. If a week passes and your menstrual period is still delayed, you should do another test after two days. If the result is still negative in the later test, there is little chance that you are pregnant. However, because there could be health-related reasons why you are not menstruating, you should see your doctor without delay.

If your menstrual cycle is irregular, it can be difficult to know the day you have missed your period. If your test indicated you are not pregnant and your menstrual cycle does not begin within a few days, you may have miscalculated your cycle and should use a second test after two days.

LIMITATION OF THE TEST

The test is not reusable. It works only if the instructions are followed precisely. Although the test is highly accurate in detecting pregnancy, a low incidence of false results can occur. Check with your doctor if you get unexpected or inconsistent test results. Certain health conditions can also cause a false or irregular test result.

QUESTIONS & ANSWERS

Q. How does the test work?

A. When a woman is pregnant, her body produces a special pregnancy hormone known as hCG (human Chorionic Gonadotropin). This hormone appears in the urine as early as the first day of a missed menstrual cycle. A positive result indicates that your urine contains hCG and you can assume you are pregnant. A negative result means that no hCG has been detected and you can assume you are not pregnant.

Q. What if the two red lines appear, but one is much lighter than the other, am I pregnant?

A. Yes. You should assume you are pregnant.

Q. Can the result change after a certain length of times?

A. A positive test result might not change after the test is completed and several hours thereafter. However, a negative test result should not be read after 10 minutes.

Q. What should I do if the test result is negative and I still suspect that I am pregnant?

A. You may have miscalculated when your period was due. If your period does not start within a week, repeat the test. If you still get a negative result and your period has not started, you should see your doctor.

Q. What if I am uncertain about the other pink line in the "T" area?

A. It is recommended that you retest three days later using another tests.

IFU-BT-HCG-M.v3.2014

Manufactured by Atlas Medical
William James House, Cowley Road, Cambridge, CB4 0WX, UK
BIOTEST is a trademark of Bio-gro Sdn Bhd.
www.bio-test.info